

**Welch Allyn®
iExaminer™ SmartBracket™
Assembly Instructions**

Mat. No. 773481, 80027806 Ver. A
Revision date: 2021-01
©2021 Welch Allyn, Inc. All rights reserved.

hillrom.com/getstarted

Install the Welch Allyn® iExaminer™ App

1. Download the iExaminer App from the App Store® or Google Play®.
2. Open the iExaminer App.
3. Read and agree to the Terms and Conditions.
4. Create a password.

Assemble the SmartBracket

- 1 For phones with more than one camera lens, determine the primary lens:
 1. Open the iExaminer App and enter your password.
 2. In the Saved Exams screen, touch + (iOS) or + (Android™).
 3. In the Exam screen, touch **Add image.**
 4. Looking at your finger on your screen, move your finger toward the lens until it almost touches the lens. The lens your finger is hovering over is the primary lens.
- 2 Insert your phone into a smooth, flat, rigid case. Follow the illustrated steps to align the SmartBracket to the primary lens. Ensure the bracket is securely attached to the case.
- 3 Attach the SmartBracket to the otoscope.

Assemble the SmartBracket

