WelchAllyn®

Halogen Exam Light III

Table Mount Instructions

Model 48859

Table Mount


NOTE: Some exam table manufacturers supply mounting hardware for installation of the Exam Light III. Check with your table supplier for availability.

WARNING: It is the responsibility of the health facility of user to ensure that the mounting bracket is securely anchored to the wall or table, or that the mobile stand is securely assembled, and the exam light is securely fastened to the bracket or stand as directed below. Give consideration to the load that is placed upon the mount. Welch Allyn, Inc. does not accept responsibility for any installation, nor for damage or injury arising from the installation of any mounting bracket or mobile stand, regardless of the type of fasteners and technique used.

- Determine the side of the table that is most optimal for mounting the light. Consider the location of the nearest power outlet and patient traffic.

NOTE: Verify that the top of the mounting bracket is at least 10 inches (25.4 cm) below the table top cushion and that the mounting position will not interfere with the operation of the table.

- Using the mounting plate as a template, mark the hole locations on the table and drill four (4) .25 inch (.64 cm) holes in the table.
- Using four 8-32 screws with lock washers and nuts, install the mounting plate to the table as shown in the diagram.
- Mount the light box to the plate by inserting the pins on the back of the light box into the slots on the mounting plate. Let the light box drop down to position the pins into the narrow portion of the mounting slots.
- To secure the Light Box to the mounting plate, insert two (2) .50" (1.27 cm) screws through the bottom of the mounting plate and tighten into the bottom of the Light Box using a Phillips head screwdriver.

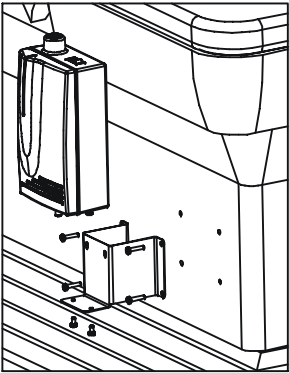- Trace the outline of the mounting plate and mark the hole locations on a 3/4 inch (1.9cm) thick piece of plywood.
- Cut the piece of wood to the outline size and drill four .25 inch (.64cm) holes.
- Using the mounting plate as a template, mark the hole locations on the table and drill (4) .25 inch (.64 cm) holes in the table.

- Using four 8-32 x 2 inch (5.08 cm) screws with lock washers and nuts, install the block of wood and mounting plate to the table as shown in the diagram.
- Mount the light box to the plate by inserting the pins on the back of the light box into the slots on the mounting plate. Let the light box drop down to position the pins into the narrow portion of the mounting slots.

- To secure the Light Box to the mounting plate, insert two (2) .50" (1.27cm) screws through the bottom of the mounting plate and tighten into the bottom of the Light Box using a phillips head screwdriver.

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